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Debonding Instructions

1. Remove all ligatures and wires.
2. Remove flash from around the brackets with a high speed debonding bur.
3. Instruct patient to bite down onto a cotton roll in the area brackets are being removed. Grasp the bracket with a straight or angled debonding plier at the bracket-adhesive interface. The bracket can be held along the occlusal-gingival or the mesial-distal edge. Gently squeeze the plier and gradually increase pressure until the bracket comes free from the tooth. **Do not twist or pull the bracket.**
4. The bracket is designed to release at the bracket adhesive interface and will leave a layer of adhesive on the enamel for safety. This means that most of the adhesive will remain on the tooth after debonding.
5. Remove remaining adhesive with a high speed debonding bur or an adhesive removing plier.
6. After bulk adhesive is removed follow up with a high or low speed enamel polishing bur.

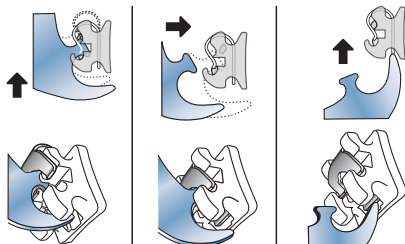
Closing

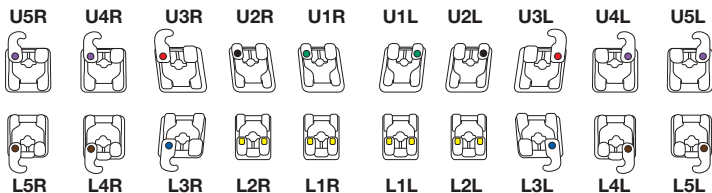
To close the *Genesis-C* Active Self-Ligating Ceramic Bracket spring clip, simply put light finger pressure on the occlusal surface of the bracket. Make sure the spring clip fully closes so the archwire is securely engaged.

Opening

Instructions For Use

To open the *Genesis-C* Active Self-Ligating Ceramic Bracket spring clip, place instrument on the open clip slot directly beneath the bracket interface and gently push in the occlusal direction to disengage the bracket clip. Then place the instrument directly in the facial opening of the clip to engage the small extension of the spring clip. Pull in the occlusal direction. The spring clip will pop open.

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