

THE Genesis™ BRACKET SYSTEM



MASEL®

The Genesis Bracket System

The *Genesis* Active Self-Ligating Bracket System and the *Genesis-C™* Ceramic Active Self-Ligating Bracket System provide low-friction passive movement and controlled active sliding mechanics, as well as an advanced-technology durable clip for reliable and predictable treatment.

Key features and benefits include:

- **Unique guide rail** for stability during clip opening and closing
- **Rounded Archwire slot** for enhanced sliding mechanics
- All brackets **open to the occlusal** to avoid tissue interferences
- **Vertical scribe line** aids in proper bracket placement along the long axis
- **Deep tie-wings** for easy optional ligation
- Interactive clip provides **passive** to **active** control for effective treatment



■ **Vertical scribe line** aids in proper bracket placement along the long axis

■ **Rounded Archwire slot** for enhanced sliding mechanics

■ Interactive clip provides **passive** to **active** control for effective treatment

■ **Unique guide rail** for stability during clip opening and closing

■ All brackets **open to the occlusal** to avoid tissue interferences

■ **Deep tie-wings** for easy optional ligation

Genesis-C Ceramic Active Self-Ligating Bracket System

- **Made from Polycrystalline Alumina** material that blends perfectly with the natural tooth tone
- **Rhodium-coated clip** for aesthetics and reliable archwire changes
- **Ceramic Injection Molding (CIM)** results in complex structures that machined brackets cannot duplicate
- **Alumina coating** provides a patterned mechanical base for predictable debonding

Genesis Active Self-Ligating Bracket System

- **Well-Polished surfaces** offers an attractive appearance
- **Rounded tie-wings and smooth ball-hooks** ensure patient comfort
- **Contoured 80-gauge mesh base** for true fit with the tooth and reliable bonding
- **Advanced technology Cobalt-Nickel clip** delivering provides reliable performance



Wire Usage

Passive Phase: Initial
Clip does not press on the wire allowing for less friction during early treatment.



Transitional Phase: Intermediate
Clip lightly interacts with the wire allowing the bracket wire interface to control tooth movement.



Active Phase: Finishing
Clip fully presses the wire into the bracket slot for precise finishing.



Tool Usage




It's easy to open the brackets with the unique *Genesis-C* or *Genesis* Opening Tools.


Opening Instructions

To open the self-ligating barcket spring clip:


1. Place instrument on the open clip slot directly beneath the bracket interface and gently push in the occlusal direction to disengage the bracket clip.

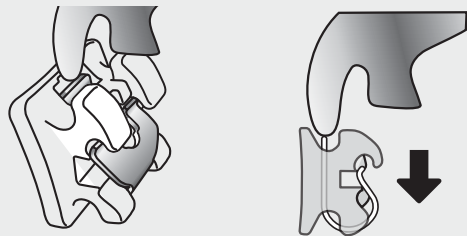

2. Then place the instrument directly in the facial opening of the clip to engage the small extension of the spring clip.

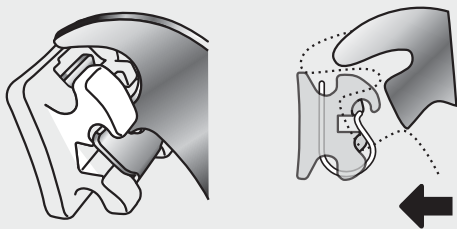

3. Pull in the occlusal direction. The spring clip will pop open. For added patient comfort, place a finger to support the tooth and this will act as a stop when opening.

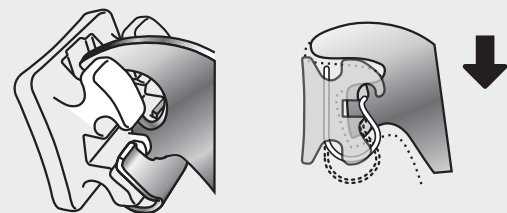

Closing Instructions

Sedit, nest, nimusda et laccusa pienisti aut qui acesto tem et faci optatur?
Quias quae et plam, optatur eriaectem etur rempor magnatusam qui ad quam
quidus moluptas et aut enis dolor si sum fugiatest ut omnihicil estia aspero ma





Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
U.S.A.
MaselOrtho.com

To Order by Phone

Domestic

Ph 888.851.0533 760.448.8600
Fax 800.888.7244 760.448.8602
Email Sales@Masel.net

International

Ph +(1) 760 448 8600
Fax +(1) 760 448 8607
Email Sales@Masel.net

Canada

Ph +(1) 800 661 9567
Fax +(1) 800 361 5088
Email cerumsup@telusplanet.net

Australia

Ph +1300 ORDERZ or +612 9540 6111
Fax (02) 9524 7623 or +612 9524 7623
Email sales@naol.com.au